

AOEL

Moisturizing HA 10



#AOEL #Hyaluronic Acid #Skinbooster

SEVENTY ONE
GLOBAL




AOEL Moisturizing HA 10

Product name	AOEL Moisturizing HA 10
Specification	3ml×5vials
Origin of country	South Korea
Shelf life	2 years
Storage condition	2°C~25°C
Effects	Moisturizing, dark spots lighten, brightening, pores tightening, anti-aging, filling fine lines
Application area	Face, neck, shoulders, back of hand, chest, inner thigh, stretch marks

Hyaluronic Acid

A substance that can be synthesized naturally in the skin. Due to its oxidizing properties, HA is a hydrophilic substance, which plays a role in providing replenish hydration and moisturizing to the skin.

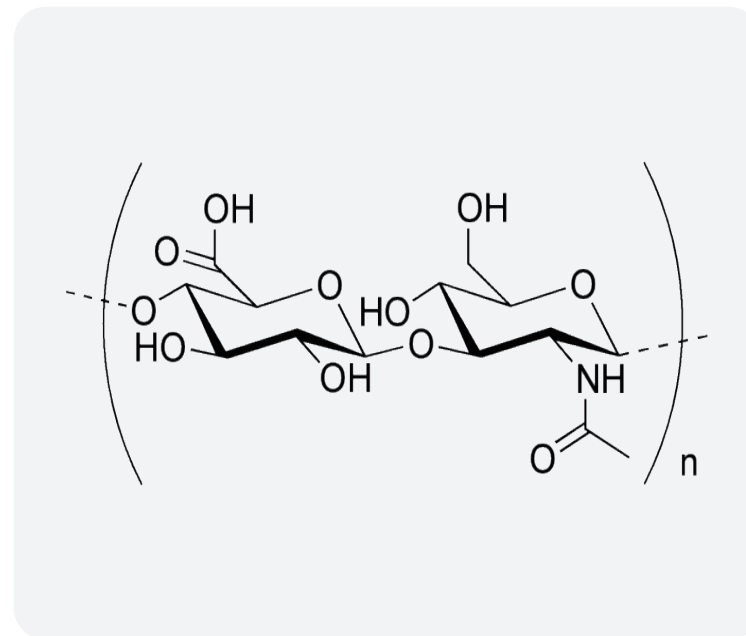


HA can absorb and store more than 1,000 times of moisture, and is also known as an indispensable main ingredient in natural moisturizers by the medical community.

HA can help skin cells fully store growth factors and nutrients, maintain a certain space between cells, and effectively prevent cell proliferation, so that the skin restores its elasticity and has a filling effect.


Human body will lose a large amount of HA as aging increases. This is the main cause of skin loss of protection, elasticity, and wrinkles.

The role of HA

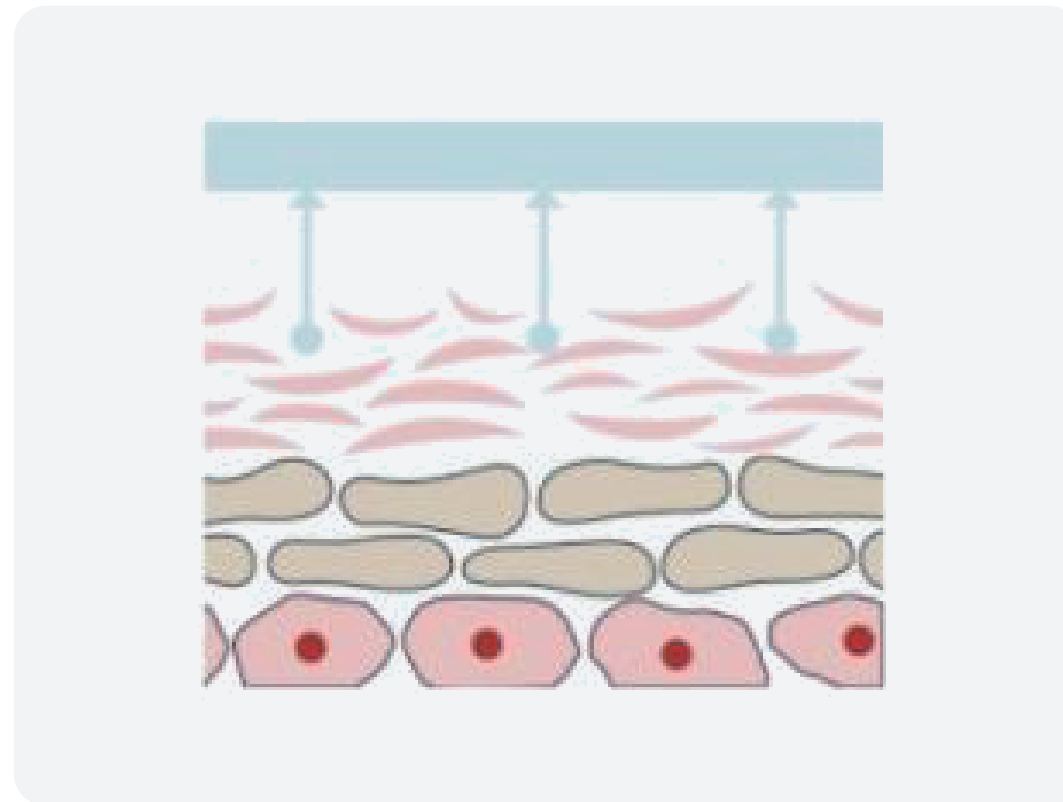

Dry damaged skin


After the skin loses moisture, the protective ability of skin decreases and it loses its elasticity.

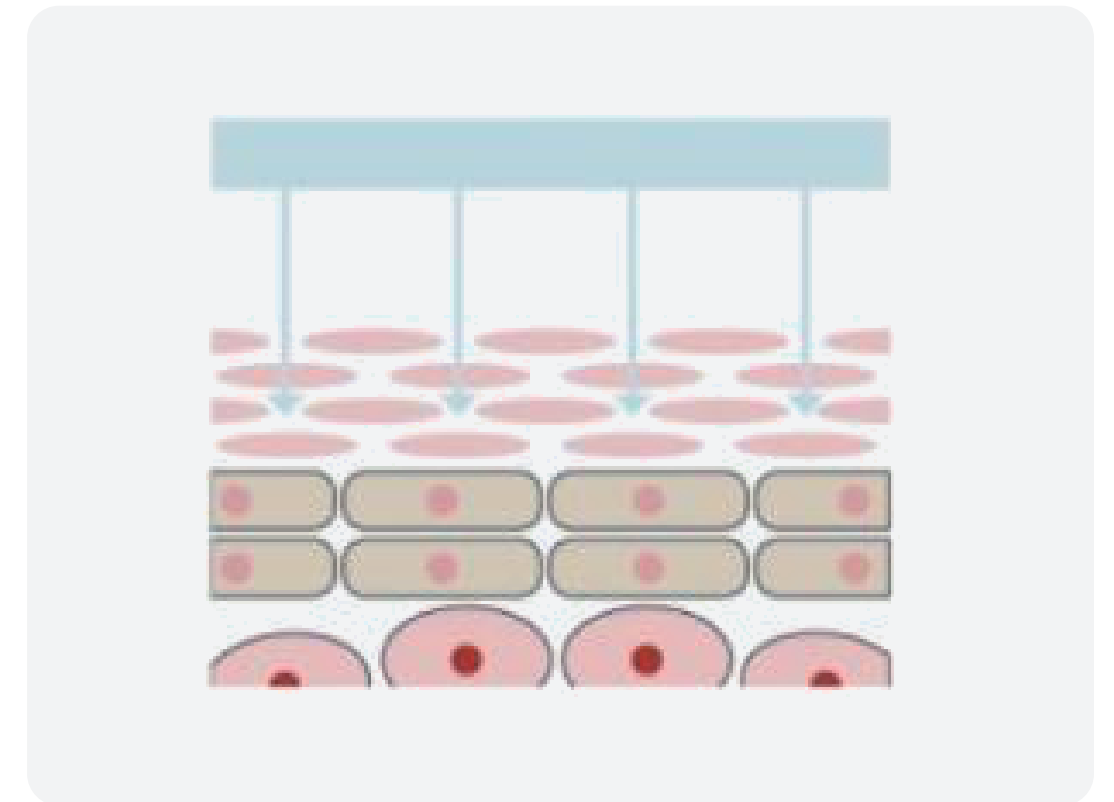


HA balances the skin

The water absorption capacity HA exceeds 1,000 times, which can replenishes the moisture under the skin and produces skin elastic.


The role of HA

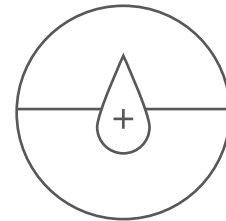


01
Effects of filling

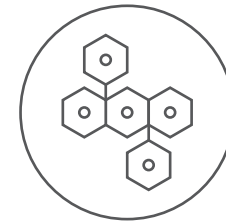


02
Maintains the structure
of skin tissue

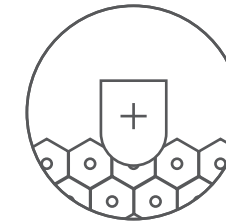


03
Protects cells

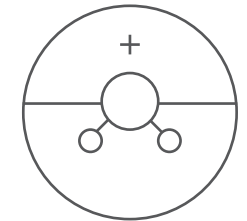


04
Replenish hydration
to the skin

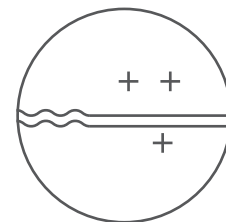

05
Maintains skin elasticity,
prevents skin from
sagging and fine wrinkles

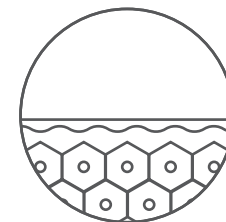


06
Dual-effects of soothing
skin for dermatitis and
sensitive skin

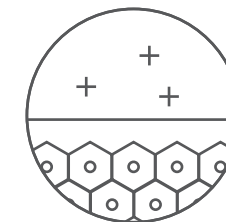

07
Maintains skin smooth
and bright skin

Helps wound healing and regeneration

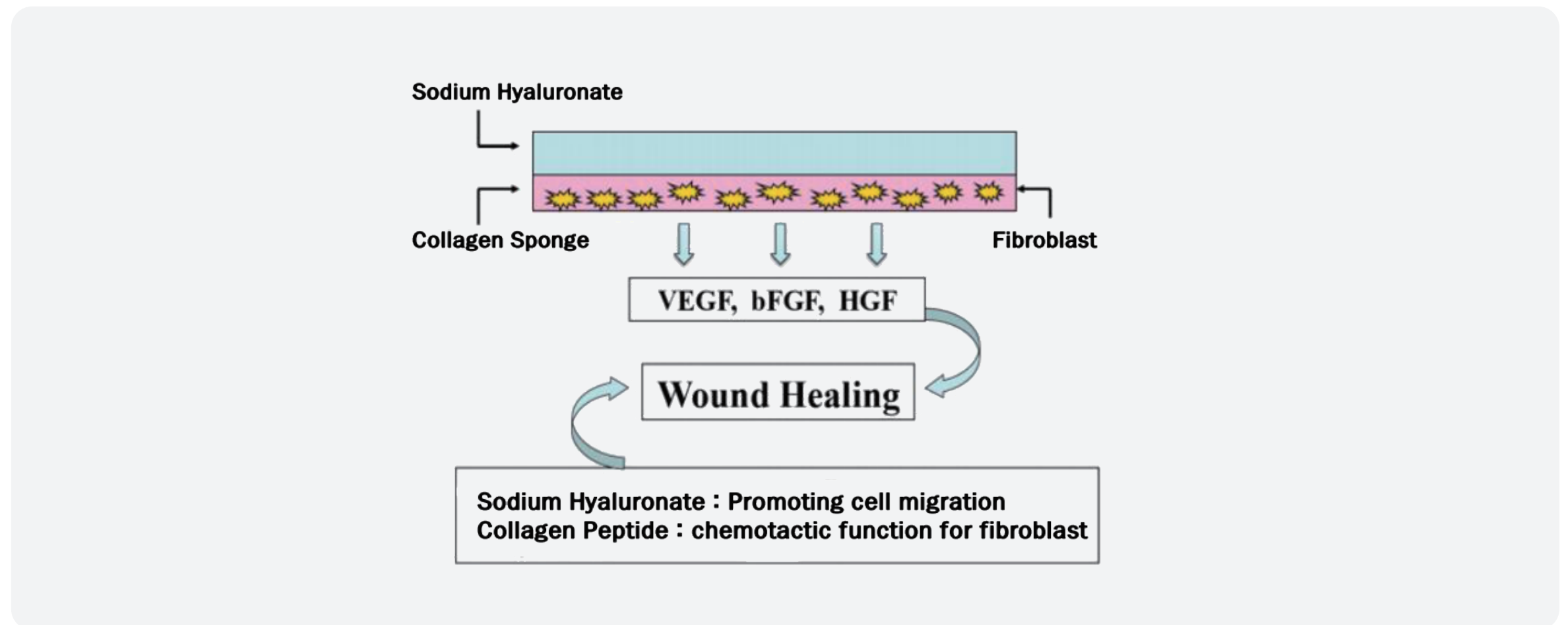


HA is an essential element to heal scars, and it is related to a variety of biological activities


It's unique water absorption, viscoelastic and rheological properties, which has more capable of wound healing.

Promote cell movement and capillary angiogenesis.


It is a component contained in the human body, and the side effects after treatment is very low.
If the effect is not satisfied or any side effects occurred after treatment, it can be dissolved and remove in time.

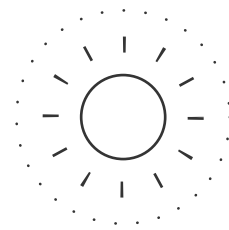


Brightening

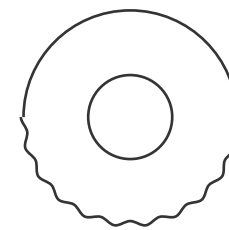

Filling facial contours
Improve wrinkles

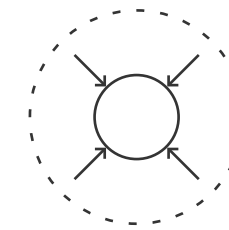

Pores tightening

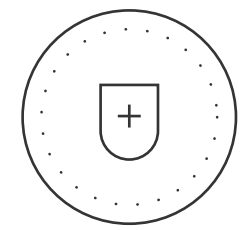

Anti- oxidation

Replenish moisture

Each molecule can absorb 218 water molecules to moisture the skin and prevent dry skin.

Dermal/subdermal contour correction

According to the different injection depth, it can correct the bump of the dermis or subcutaneous layer.

Promote collagen regeneration in the dermis

Effectively stimulate fibroblasts and promote collagen regeneration at application area.

Antioxidant

After removing free radicals caused by ultraviolet rays, it blocks the polysaccharide chains, and decomposes antioxidant substances.



ADEL

Moisturizing
HA 10

Age Of Elegance

- Sodium Hyaluronate
- Sodium Chloride

3 ml x 5 vials



Injection manual

Needle selection

30G/34G 4mm needle

Injection method

45° intradermal injection, 3mm injection depth, 0.5~1cm interval point, 0.02~0.03cc each point

Recommend dosage

4~6ml for the whole face

Application cycle

1. 3 times as a course of treatment, every 15 days to 1 month interval time
2. Once every 4 weeks for initial stage, then once every 2 to 3 months in the second year
3. 3 weeks after first application, then 6 weeks interval time, and then 9 weeks interval time

Application area

Face, neck, shoulders, back of hand, chest, inner thigh, stretch marks

**MTS application
Procedure****MTS application method**

1. Clean face
2. Apply anesthetic cream for about 30 minutes
3. Use alcohol cotton swab to remove anesthetic cream, and disinfection whole face
4. Use MTS micro needle roller (0.5mm~1mm) to first roll vertically, then horizontally, and finally roll obliquely until the skin becomes red. If there is bleeding, disinfect with alcohol cotton swab in time
5. Gently apply product from ampoule on the skin
6. After MTS, you may apply a medical sterile soothing sheet mask to help reduce skin redness as soon as possible

Application Cycle

10~14 days once, 3 times as a course of treatment

Basic combination

Combination option	Ratio	Effects
HA10	1	Moisturizing, brightening, pores tightening, anti-oxidant, filling fine wrinkles
HA10+Skin Booster	1:1	Moisturizing, brightening, pores tightening, whitening, dark spots lighten, anti-oxidant, promoting regeneration of collagen, and providing necessary nutrients to skin, etc.
HA10+Rejuvenating	1:1	Moisturizing, brightening, whitening, dark spots lightening, anti-oxidation, pores tightening, repairing acne scars, improving skin elasticity, promoting cells regeneration, etc.
HA10+Skin Booster +Rejuvenating	1:1:1	Moisturizing, brightening, whitening, dark spots lightening, anti-oxidation, pores tightening, repairing acne scars, promoting cells regeneration, synthesizing collagen, improving skin elasticity, reducing fine wrinkles, enhancing firmness, and providing necessary nutrients to skin. It's a comprehensive anti-aging combination

Brightening ageless combo (Upgraded professional edition)

Level	Ratio	Effects
●	TheraFill 0.5cc+ Saline 1cc+Toxin12U +Skin Booster 0.2cc or Rejuvenating 0.2cc	skin brightening, whitening, promoting regeneration of collagen, filling wrinkles and fine lines, enhancing firmness. Effectiveness can be seen at least a month and can be maintained longer, comparing the general skin booster combination, and better in improving skin elasticity.
● ●	TheraFill 0.5cc+ Saline 1cc+Toxin12U +Skin Booster 0.2cc or Rejuvenating 0.2cc	
● ● ●	TheraFill 0.5cc+ Saline 1cc+Toxin12U +Skin Booster 0.2cc or Rejuvenating 0.2cc	
● ● ● ●	TheraFill 0.5cc+ Saline 1cc+Toxin12U +Skin Booster 0.2cc or Rejuvenating 0.2cc	

Precautions for use

Before treatment

Avoid menstruation period and avoid aggravating bruises and swelling

Do not take anticoagulants before treatment, to avoid bruising and bleeding at the injection site

Remove makeup and keep face clean

If you have severe allergies, pimples, acne or wounds, please consult a doctor to avoid inflammation occurs

Confirmation of wounds, infections, and allergies before treatment

Confirmation of fever, flu, chronic diseases and infectious diseases before treatment

Patient should inform in advance if have undergone facial cosmetic surgery, laser surgery, and implantation of any medical objects

After treatment

Keep the treatment area dry, avoid touching the treatment area with hands, and stay away from water for 24 hours to prevent infection

Avoid eating spicy and heavy food for 3 days, avoid smoking and alcohol consumption

Stay away from direct sunlights, do not make up for a short time, nor use substantial skin care products

Apply medical sterile repair mask every night for 3 days

Avoid saunas, strenuous exercise or hypercaloric therapy or any activities that may cause sweating and fever within a week

Do not use ointments, skin care products, and make-up at treatment area after surgery to reduce the infection

Maintain a resting schedule and drink plenty of water

Recovery period

There may be bruises and redness after the treatment, which may occurs individually, It will disappears within 4-7 days usually, and the effects begin to appear during the recovery period



ADEL

Moisturizing
HA 10

Age Of Elegance

- Sodium Hyaluronate
- Sodium Chloride

3 ml x 5 vials





ADEL
Skin
Booster
Ampoule
3ml

ADEL
Body Solution
Essence
10ml

ADEL
Skin
Rejuvenating
Ampoule
3ml

ADEL
Moisturizing
HA 10
3ml

ADEL
Face Solution
Essence
10ml

Thank you

SEVENTY ONE
GLOBAL

www.71global.co.kr